

# QAL Medical™ OrthoAgility®

## OrthoAgility™

Immediately remove and dispose of the Patient Kit.

QAL Medical devices are constructed with components made from aluminum (silver), stainless steel (silver), polycarbonate plastic (blue), modified polyphenylene oxide plastic (light blue, trade name NORYL), acetal plastic (white), polypropylene plastic (translucent white), polyethylene plastic (translucent white) and nylon web straps (black). These materials offer excellent resistance to common cleaning materials such as soaps and mild cleaning agents when used as recommended.

1. Use a cloth and a mild soap solution to remove dirt and deposits.
2. Wipe soap deposits from the equipment using a clean, damp cloth.
3. For stubborn areas, use a household spray cleaner applied to a soft cloth. After cleaning, wipe off residue immediately with a water-dampened cloth.
4. Do not pour cleaning solutions directly onto the machine. This may allow fluids to enter the machine and cause electrical problems, or wash lubricants away from the running components, reducing the life span of the device.
5. NEVER IMMERSE THE DEVICE IN FLUIDS.
6. Patient Kits (soft goods) for the device are for single patient use only and cannot be washed for reuse.

The clean CPM device must be equipped with a new patient kit prior to use by the next patient.